TE201 User Manual

Suitable for Model: CZ-B501

Contents

Precautions -----	01
Product Introduction -----	01
Functional Operation Instructions -----	02
Specifications -----	04
Troubleshooting -----	05
Package Includes -----	06
CAUTION -----	07
Guarantee -----	76

EN(1-9)

DE(10-22)

FR(23-33)

IT(34-44)

ES(45-55)

RU(56-66)

JP(67-75)

EC

REP

Germany Retevis Technology GmbH
Uetzenäcker 29, 38176 Wendeburg
Retevis-Europa@outlook.com
Tel:+0049 053029369179

Precautions

- 1、 This device (hereinafter referred to as "the device") is a sophisticated and sensitive audio amplification electronic device. Regularly clean earwax and debris from the surface using a dry or soft cloth—avoid solvents or cleaning agents.
- 2、 When not in use, store the earbuds in the charging dock to prevent continuous operation, prolong battery life, and reduce power consumption.
- 3、 Keep the device out of reach of children to prevent accidental ingestion or damage.
- 4、 Store in a dry environment and wipe charging contacts before each charge to minimize malfunctions.
- 5、 Remove the device before bathing, sleeping, or swimming to prevent moisture damage. Do not clean with water or other liquids.

Product Introduction

No.	Display	Function Description
1	M Button	Power On/Off
2	+ Button	Volume Up (6 levels)
3	- Button	Volume Down (6 levels)
4	Left Charging Slot	Headphone Charging Slot
5	Left Battery Indicator	Displays left earbud battery level
6	Earbuds	Replaceable Earbuds
7	Base Battery Indicator	Displays charging base battery level
8	Right Charging Slot	Headphone Charging Slot
9	Right Battery Indicator	Displays right earbud battery level
10	Type-C	Supports charging via Type-C cable

Functional Operation Instructions

3.1 Power On/Off

- ①Place the amplifier into the charging base. After 5 seconds, the amplifier will automatically turn on. Ensure the amplifier is picked up and placed in the ear within 5 seconds to avoid unnecessary feedback;
- ②When the amplifier is on, long-press the M button to turn it off. When the amplifier is off and not on the charging base, long-press the M button to turn it on;
- ③When the amplifier is placed in the charging base for 5 seconds, it will automatically turn on.

3.2 Charging & Battery Status

①Insert the amplifier into the charging base. The charging base's digital screen will display the current battery level of the base and the charging status of the amplifier. The screen will automatically turn off after about 10 seconds.

②When the amplifier is removed from the charging base, the base will display the remaining battery level.

Note: The charging base can support approximately 4 full charges for the amplifier.

3.3 Volume Adjustment

①After powering on, press the "+" button briefly to increase the volume. A long beep indicates the maximum volume level;

②After powering on, press the "-" button briefly to decrease the volume. Three beeps indicate the minimum volume level;

Note: A short beep will sound with each volume adjustment.

3.4 Mode Switching

①While powered on, press the M button briefly to cycle through listening modes. The default mode is Standard Mode.

②One Beep: Indoor Mode;

③Two Beeps: Outdoor Mode;

④Three Beeps: Cinema Mode;

⑤Four Beeps: General Standard Mode;

⑥The amplifier will remember the last mode set before shutdown.

3.5 Low Battery Alert

When the battery level drops below 15% and the voltage is less than 3.4V, a low battery alert ("three beeps") will sound. The amplifier will shut down when the voltage drops below 3.3V.

Specifications

Parameter	Specification
Battery Type	Built-in polymer rechargeable lithium battery
Battery Capacity	Amplifier:LIR854,3.6V/25mAH,0.9g Charging case:GX902030,3.6V/500mAH, 9.78g Note:usable extinguishing agent:water, foam,CO2
Charging Time	1 ~ 2 hours
Charging Base Input	DC 5V / 1A
Amplifier Input	DC 4.2V / 100mA
Maximum Output Power (OSPL90)	$\leq 115\text{dB} + 3\text{dB}$
Saturation Sound Pressure (OSPL90)	MAX: 110dB
Full Gain Sound Gain (Average)	MAX: $45\text{dB} \pm 5\text{dB}$

Equivalent Input Noise	$\leq 20\text{dB} \pm 3\text{dB}$
Total Harmonic Distortion	$\leq 5\%$
Response Frequency Range (Not Narrower Than)	300Hz---5000Hz
Operating Current	2.5mA
Rated Power Consumption	10mA
Shutdown Power	15uA
Standby Power	25uA
Operating Temperature	0°C ~ 40 °C
Operating Humidity	$\leq 80\%$

Troubleshooting

If your amplifier earphones encounter issues during use, please refer to the table below for initial troubleshooting steps or contact the seller for specific repair advice.

Fault Phenomenon	Possible Causes	Solutions
Noise	①Low battery power ②Damaged volume buttons ③Poor battery contact	①Replace the battery ②Contact us ③Contact us

Feedback	①Excessive earwax or dirty earbuds; ②Incorrect earbud placement ③High volume ④Poor earbud seal	①Excessive earwax or dirty earbuds; ②Reinsert the earbuds ③Lower the volume ④Replace with appropriately sized earbuds
No Sound	①Dead battery ②Blocked sound outlet ③Damaged volume buttons ④Poor battery contact	①Replace the battery ②Clean the sound outlet ③Contact us ④Contact us

Note: Do not disassemble the product or visit unauthorized repair shops to avoid potential damage. Please consult the seller for specific handling methods.

Package Includes

No.	Item	Function	Quantity	note
1	Charging Base	Charge the amplifier	1pcs	
2	Charging Cable	Charge the charging base	1pcs	
3	Storage Bag	Store amplifier	1pcs	
4	Earbuds	Fit amplifier	3 sets (S/M/L)	Choose based on comfort;
5	User Manual	Guides usage	1pcs	

Simple EU declaration of conformity

• HONG KONG RETEKESSE CO., LIMITED declares that the radio equipment type is in compliance with the essential requirements and other relevant provisions of RED Directive 2014/53/EU and the ROHS Directive 2011/65/EU and the WEEE Directive 2012/19/EU; the full text of the EU declaration of conformity is available at the following internet address: www.retekess.com.

Disposal

The crossed-out wheeled-bin symbol on your product, literature, or packaging reminds you that all electrical and electronic products, batteries, or accumulators must be taken to designated collection locations at the end of their working life. Do not dispose of these products as unsorted municipal waste. Dispose of them according to the laws and rules in your area.

Electromagnetic Interference/Compatibility

Nearly every electronic device is susceptible to electromagnetic interference (EMI) if inadequately shielded, designed, or otherwise configured for electromagnetic compatibility. During transmissions, your radio generates RF energy that can possibly cause interference with other devices or systems. To avoid such interference, turn off the radio in areas where signs are posted to do so, such as hospitals or healthcare facilities.

Protect your hearing

- Use the lowest volume necessary to do your job. Turn up the

volume only if you are in noisy surroundings.

- Limit the amount of time you use headsets or earpieces at high volume.

- When using the radio without a headset or earpiece, do not place the radio's speaker directly against your ear.

- Use carefully with the earphone maybe possible excessive sound pressure from earphones and headphones can cause hearing loss.

CAUTION: Exposure to loud noises from any source for extended periods of time may temporarily or permanently affect your hearing. The louder the radio's volume, the less time is required before your hearing could be affected. Hearing damage from loud noise is sometimes undetectable at first and can have a cumulative effect.

Battery Safety

- Turn off your radio before charging.
- Batteries shall be charged at 10°C~40°C environment temperature specified. If the environment temperature is lower than 0°C, the charge shall be prohibited.
- The plug of the adapter is considered a disconnect device. The socket-outlet shall be installed near the equipment and shall be easily accessible.
- Rechargeable batteries are only to be charged by adults.

Children's products

WARNING: CHOKING HAZARD-Small Parts. Not suitable for children under 3 years old.

Adapter

Equipped with Type-C Charging Port

The power delivered by the charger must be between min [0.42] Watts and max [2.1] Watts required by the radio equipment in order to achieve the maximum

Authorized Accessories List

- Contact Reteless for assistance regarding repairs and service.
- For a list of Reteless-approved accessories for your radio model, visit the website <http://www.reteless.com>

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Post Code: _____ Email: _____

Remarks:

1. This guarantee card should be kept by the user, no replacement if lost.
 2. Most new products carry a two-year manufacturer's warranty from the date of purchase.
 3. The user can get warranty and after-sales service as below:
 - Contact the seller where you buy.
 - Products Repaired by Our Local Repair Center
 4. For warranty service, you will need to provide a receipt proof of purchase from the actual seller for verification
- Exclusions from Warranty Coverage:
1. To any product damaged by accident.
 2. In the event of misuse or abuse of the product or as a result of unauthorized alterations or repairs.
 3. If the serial number has been altered, defaced, or removed.

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